# INSTRUCTION FOR USE

Medical compression stockings

- |   |   |  |
|---|---|--|
| <b>(EN)</b> Medical compression stockings     | <b>(NL)</b> Medische compressiekousen                 | <b>(SL)</b> Medicinske kompresijske nogavice |
| <b>(DE)</b> Medizinische Kompressionsstrümpfe | <b>(PT)</b> Meias de compressão medicinais            | <b>(RU)</b> Медицинские компрессионные чулки |
| <b>(FR)</b> Bas de compression médicale       | <b>(PL)</b> Medyczne pończosznicze wyroby kompresyjne | <b>(IS)</b> Teygjusokkar til lækninga        |
| <b>(ES)</b> Medias de compresión médica       | <b>(HU)</b> Kompressziós térdharisnya                 | <b>(AR)</b> جوارب طبية ضاغطة                 |
| <b>(SV)</b> Medicinska kompressionsstrumpor   | <b>(IT)</b> Calze compressive medicali                | <b>(TR)</b> Tibbi kompresyon çorapları       |
| <b>(NO)</b> Medisinske kompresjonsstrømper    | <b>(FI)</b> Lääkinnälliset kompressiosukat            | <b>(HE)</b> גרבי לחץ רפואיים                 |
| <b>(DA)</b> Medicinsk kompressionsstrømpe     | <b>(RO)</b> Ciorapi compresivi medicali               |  |

<b>(EN)</b> FIBER CONTENT	<b>(HU)</b> ROSTTARTALOM
<b>(DE)</b> MATERIALZUSAMMENSETZUNG	<b>(IT)</b> COMPOSIZIONE DEL MATERIALE
<b>(FR)</b> COMPOSITION	<b>(FI)</b> KUITUSISÄLTÖ
<b>(ES)</b> CONTENIDO DE FIBRA	<b>(RO)</b> CONTIN FIBRE
<b>(SV)</b> FIBERINNEHÅLL	<b>(SL)</b> VSEBNOST VLAKEN
<b>(NO)</b> FIBERINNHOLD	<b>(RU)</b> СОСТАВ ВОЛОКОН
<b>(DA)</b> FIBERINDHOLD	<b>(IS)</b> TREFJAINNIHALD
<b>(NL)</b> VEZELGEHALTE	<b>(AR)</b> محتوى النسيج
<b>(PT)</b> TEOR DE FIBRA	<b>(TR)</b> FIBER İÇERIKLI
<b>(PL)</b> ZAWARTOŚĆ WŁÓKIEŃ	<b>(HE)</b> תכולת הסיב

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## Dear Customer,

Thank you for choosing JOBST®, the medical compression stockings that incorporate physician recommended gradient compression for the management of venous and lymphatic diseases. JOBST® products are designed to relieve tired, aching legs, help treat venous and lymphatic conditions, and help prevent swelling of feet, ankles and legs.

### Comfort

Because it is important to wear them every day, your JOBST® medical compression stockings are designed with all-day comfort in mind.

### Health

According to your physician's recommendation your JOBST® medical compression stockings apply a specific amount of compression to your legs. This is called gradient compression because it is highest at the ankle and tapers off up the leg. This helps promote the blood flow in your legs. A wide range of compression levels is available to manage different venous and lymphatic conditions relieving tired, aching legs, pain and discomfort.

### Style

Only you will know you are wearing medical compression stockings. You can use your JOBST® product with any of your clothes for any occasion.

## 1. Application

### Tips for Wearing JOBST® Medical Compression Stockings:

To help your legs look and feel good, wear your JOBST® medical compression stockings regularly and consistently. Ideally you should put on your compression stockings in the morning, immediately after getting up. You may experience some difficulty putting on the stockings in the beginning but this will become easier after a number of days. When applying the compression stockings, please do not pull on the edge of the stockings but rather follow these recommendations:

Turn the compression stocking inside out all the way to the heel.

Open the foot fabric using both hands and pull over the foot all the way to the heel.

Gradually pull the compression stocking over instep and heel.

Then gradually pull the compression stocking upward by returning the stocking to a right-side out position piece-by-piece.

Then distribute the stocking evenly by massaging with both hands until the stocking fits without forming creases. It is particularly helpful to wear rubber gloves for this step.

## 2. Indications

Every JOBST® product has a specific compression level designed to help manage specific venous and lymphatic conditions.

### 15-20 mmHg\*

Tired, aching legs; minor ankle, leg and foot swelling; minor varicosis.

### 20-30mmHg\*

Tired, aching legs; moderate to severe varicosis; post vein surgery; post sclerotherapy; moderate tendency of edema; for adjunct management of superficial thrombophlebitis/SVT; helps prevent recurrence of ulcerations.

### 30-40mmHg\*

Severe varicose veins; edema; active patients with lymphoedema in ISL stage I-III with no or minimal shape distortion (not applicable for Casual Pattern); post vein surgery; post sclerotherapy; venous ulcers; CVI; manages manifestations of PTS; helps prevention of post thrombotic syndrome.

\*The mean compression for an average ankle size.

## 3. Contraindications

**JOBST® medical compression stockings SHOULD NOT BE WORN and is contraindicated if you have any of the following conditions:**

Ischemia (e.g. advanced arterial disease); uncontrolled congestive heart failure; untreated septic phlebitis; Phlegmasia cerulea dolens; incompatibility to fabric

## 4. Cautions

**Consult your physician for advice if you have or experience:**

Skin infections; weeping dermatoses; advanced peripheral neuropathy; severely compromised skin sensibility and impaired sensitivity of the limb; immobility (confinement to bed).

Conditions in which increased venous and lymphatic return is not desired.

Appropriate hygiene and skin care should be applied.

The attending physician decides whether compression in general can be applied to patients suffering from the following diseases: rheumatoid arthritis, complex regional pain syndrome, CRPS (M. Sudeck), malignant lymphedema, gangrene.

Incorrectly measured and badly fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage. Open

wounds should be covered with an appropriate dressing before the compression garment is put on.

Appropriate footwear must be worn when walking to help prevent falls.

Medical compression garments should only be used after consultation with your doctor or therapist and checked regularly by them on the appropriateness of your therapy, also considering age and health status.

Patient measurements should be taken by a trained person to determine the correct product size needed and to decide whether a made to measure or ready-to-wear garment is required.

Ill-fitting compression garments may cause constriction leading to e.g. skin necrosis, pressure ulcers and pressure related peripheral nerve damage.

When re-ordering a garment after 6 months we recommend remeasuring the patient as the limb circumference might have changed.

If you notice discomfort, pain, numbness, changes in skin color, appearance of new wounds or swelling during wear, stop using the product and contact your healthcare professional.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of your state.

## 5. Care instructions

Machine washable in warm water. Do not bleach. Tumble dry - low heat. Do not dry clean. Do not iron. Use a delicate fabric detergent. Do not use fabric softener. Rinse well. Do not wring.

Please protect your compression garments from direct exposure to sunlight, heat, and moisture. We recommend storage at room temperature.

## Silicone band care:

When needed, hand wash silicone band to remove excess creams, lotions, hair and other foreign material.

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BSN medical Inc.
BSN medical GmbH
Australian Distributor: BSN medical (Aust.) Pty Ltd.
Importer for the European Union: BSN-Jobst GmbH