JoViPak

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BENEFITS

Custom Leg:

The stitched lines of JoViPak are necessary to keep the foam particles held in place and follow the pattern design providing patient comfort, softening of fibrotic tissue and movement of lymphatic fluid.

Custom Leg JoViJacket:

Provides compression and support for overnight containment of swelling.

Custom Zipper:

Increases ease of donning for elderly patients or those with low mobility.

INDICATIONS

Use nightly for lymphedema or edema in the lower extremity.

Custom Zipper Ankle to Knee:

Required when there is a 10 cm or more difference in the Heel to Ankle (H/A) and Ankle (B) circumferences.

Custom Zipper Knee to Groin:

Increases ease of donning for elderly patients or those with low mobility.

PRECAUTIONS

Patients with congestive heart failure should have physician approval. Be aware of open wounds or burns from recently radiated skin. Observe the patient with fresh or open wounds, congestive heart failure, or freshly radiated skin. Open wounds must be covered with an appropriate dressing before donning the garment.

CONTRAINDICATION

Do not use in the presence of uncontrolled congestive heart failure, untreated cancer in affected area, deep vein thrombosis, absent or severely impaired sensation, sepsis, infected wounds or tissue, or if sensitive to the materials in the garment.

WARRANTY

Custom Leg Sleeve and Custom Zipper:

Guaranteed against defects in material and craftsmanship for up to 6 months (if made of Cotton) to 12 months (if made of Powerdry) with normal use and care.

Custom Leg Sleeve JoViJacket:

Guaranteed against defects in materials and craftsmanship for 3 months with normal use and care. Replace every 6 months.

DONNING INSTRUCTION

Roll the top of the JoViPak garment back onto itself and slide your foot into the garment as far as possible. Wear the JoViJacket on top of the garment.

CARE INSTRUCTIONS

Machine wash / Gentle
Mild detergent
Warm water / No bleach
Machine dry / Low temp.

JoViJacket:

Hand wash / Warm water
Mild detergent
No bleach / Air dry

CARE

Gentle wash cycle 30° C

Tumble drying is possible,
low temperature

Do not bleach

Do not iron

Do not dry clean

CARE

JoViJacket

Maximum hand-washing temperature 40° C

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© 2017 JoVIPak Corp.